
**10/ INR measurement**  
The default value of the measurement is the INR value. Please refer to the LabPad® user guide "Measurements" section.

To inject the Tsmart™ INR, place the LabPad® upside down and push the side blue button. Discard it preferably in a bin dedicated to biological waste. (see the LabPad® user guide: "How to discard the microcuvette section").

**11/ Results**  
INR value ranges from 0.8 to 8. Prothrombin Time (PT) between 7.2 and 72 seconds. Quick Time (QT) between 10 and 110%. If the result is out of range, an error message is displayed; in this event, refer to the LabPad® user guide "Error messages" section.

**12/ Cleaning and disinfection instructions**  
Once the microcuvette is taken from its pouch, it can be used immediately. Provided storage and operating conditions are fulfilled (see sections 6 & 7).  
For cleaning the device, please refer to the LabPad® user guide "Cleaning" section.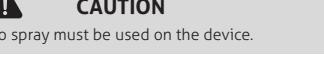

**CAUTION**  
No spray must be used on the device.

**13/ Quality Controls**  
The LabPad® testing device is a technologically advanced system that automatically performs the test required before starting the measurement. Various controls are performed during the test to ensure that the measurement occurs. An error message is displayed in the device screen.  
(see the LabPad® user guide "Error messages" section).

**14/ Performance characteristics**  
INR CV results are precise within 6% using capillary blood and +5% using venous blood in the therapeutic range. These performance data are based on several clinical studies conducted on capillary blood and venous blood in comparison with a laboratory reference as well as the WHO INR-tube method. They are available for healthcare professionals only, upon request.

**15/ Test limitations and interferences**

- Bilirubina up to 513 µmol/L (30/dL) between 25 and 55%
- Hematocrite up to 1.000 mg/dL (Hämoglobin)
- Triglycerides up to 11.3 mmol/L (1.000 mg/dL)

Use of an alternative method of measurement is recommended in the event of a transition period with a heparinized treatment.

The clotting factor sensitivity for Factors II, V, VII and X has been evaluated. Data are available for healthcare professionals only, upon request.

**3/ Test principle**  
Tsmart™ INR is a single-use microcuvette integrating innovative microtechnology containing a procoagulant reagent embedded in a dried form. Once inserted into the LabPad® testing device, a capillary blood sample from a fingertip is dropped on the microcuvette and mixed with the reagent allowing the automatic quantitative measurement of the Prothrombin Time (PT), International Normalized Ratio (INR) and Quick Time (QT). This measurement is necessary for patients under oral anticoagulant treatment with Vitamin K Antagonists (VKA) to regularly check their INR. A linear correlation exists between INR and Prothrombin Time which depends on the biological parameters of the microcuvettes' lot. Before every measurement, these parameters are read from the microcuvette's datamatrix.

**4/ Contents of the microcuvette**  
Each microcuvette contains a reagent that gives its whitish color to the central micro channel. It contains a human recombinant thromboplastin to which adjuvants and a heparin inhibitor have been added.

**5/ Cautions and warnings**  
5.1 **Storage**  
Before use, check the expiry date on both the box and the individual pouch. In case of doubt, contact your reseller. Likewise, the lot number printed on the individual pouch must be identical to the one printed on the box. If this is not the case, contact your reseller. Do not forget to insert a microcuvette in the device LabPad®. You prick the finger, do not prick the blood sample directly in the LabPad®.

**5.2 Warnings**  
The Tsmart™ INR is linked to the action of the VKA oral anticoagulants but this action may be modified in the event of interaction with other drugs taken simultaneously. Similarly, some changes in diet and certain disorders can have an impact on this anticoagulant action.

**6/ Storage and handling**  
Store the Tsmart™ INR microcuvettes in their individual pouch until they are used. You can store them between 15° and 25° until the expiry date printed on the pouch.

**7/ Operating conditions**  
Place the LabPad® on a stable, flat and vibration-free surface. The ambient temperature must be between 15° and 32° and the humidity level less than 85%.

The Tsmart™ INR microcuvette must also be at room temperature (see section 6). The microcuvette is sensitive to humidity; if the microcuvette is sensitive to humidity, you are advised to refrain from using the Tsmart™ INR under high humidity conditions. Once the pouch is open, the microcuvette should be used within 10min.

**8/ Getting ready**  
Please refer to the LabPad® user guide.  
Check that all operating conditions are fulfilled (see section 7 Operating conditions) and that you have prepared all the need for the test:

- a single-use lancet or a lancet device with a new needle.
- Check that they are adapted to INR measurement; a 21G lancet is recommended. Please refer to the manufacturer's instructions for use.
- dressing gauze or a paper tissue
- a bandage

**9/ Collecting a blood sample**  
Wash hands of the patient. Cleanse the skin with soap and dry them thoroughly. Insert the Tsmart™ microcuvette in the LabPad®. You have two minutes to prick the finger and drop the blood sample when the user interface ask you. Use the lancet or the lancet device to prick the side of the fingertip of the non dominant hand. If the lancet or the lancet device is not available, clean a blood sample with a volume of 5µl of blood is required to fill the central micro channel. Do not squeeze the finger of the patient, as this may lead to incorrect results.

The required blood drop should be deposited within 15 seconds after pricking as the natural coagulation process will have already begun. When you drop the blood, be careful not to touch or hold the microcuvette, and not to spread the blood on the microcuvette's plate (see the LabPad® user guide). Make sure you fill the microcuvette with the Tsmart™ INR under high humidity conditions. Once the finger is open, the microcuvette should be used within 10min.

If you decide to drop a second sample of blood, discard the microcuvette and start again by prick another finger. Do not add blood once the measurement has begun if necessary, wipe the excess blood from the finger of the patient and apply the bandage.

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